



August 19, 2021

Subject: Cleaning Products to be used with the enFlow™ IV Fluid/Blood Warming System

Dear Valued Customer:

When cleaning the enFlow IV Fluid/Blood Warming System please be aware of the cleaning products used. The following list of approved cleaning solutions may be used to clean the Warmer and Controller:

- 1.3 mL of Dawn® Ultra Dishwashing Liquid diluted in 1 liter of 80 – 90 ° F water
- CaviWipes®
- 4% Hydrogen Peroxide
- CIDEX OPA

Other acceptable cleaners are:

- 40% bleach sodium hypochlorite and 60% water solution
- Isopropyl Alcohol
- Clorox Healthcare Bleach Germicidal Wipe
- Cidex OPA
- Cidex Activated Glutaraldehyde Solution
- Cavi Wipes
- Super Sani-Cloth Germicidal Disposable Wipe
- Sani-Cloth HB Germicidal Disposable Wipe
- Sani-Cloth AF3 Germicidal Disposable Wipe

Consult your enFlow Operator's Manual for a full description of the cleaning and disinfecting procedure for the enFlow device. Please contact your local Vyairé Medical sales consultant with any additional questions.

Kind Regards,

Vyairé Medical

26125 N. Riverwoods Blvd.
Mettawa, IL. 60045
+1 833 327 3284
vyaire.com

AS WE LIVE AND BREATHE