



May 19, 2021

Dear Valued Customer,

Vyaire would like to thank you for your patience and continued trust in Vyaire products during the redesign of the enFlow™ Cartridge.

The redesigned cartridge features a parylene coating, a widely used material in medical devices, protecting the fluid it warms from the equipment itself. This design has been extensively tested. The results indicate biological safety with the use of a variety of intravenous solutions in different therapeutic scenarios.

The “keying aspect” of the product has been updated to ensure that only the new warmer, redesigned to accommodate this new key, and new cartridge will work together. (Fig. 1)
There have been no changes to the enFlow controller, this is compatible with the updated warmer.

Figure 1. Parylene coated disposable cartridge and warmer have square shape in keying. Circled in red below.



Please contact your Vyaire sales representative with questions and for ordering information.

Kind regards,

Vyaire Medical

26125 N. Riverwoods Blvd.
Mettawa, IL. 60045
+1 833 327 3284
vyaire.com

AS WE LIVE AND BREATHE

For EU, Australia and Canada distribution.

© 2021 Vyaire. Vyaire, the Vyaire logo and all other trademarks or registered trademarks are property of Vyaire Medical, Inc., or one of its affiliates. VYR-GBL-2100117